



## Cleaning Reusable Cables

### Instructions for Cleaning Catheter End of Cable

As the need for cleaning arises, we recommend that the cables be wiped off with soap and water and swabbed, if necessary. Liquid sterilants may be used but should be rinsed off afterward. The Koala cables may also be gas sterilized.

The following cable cleaning instructions are included with the cable:



### Cable Connector Cleaning Kit

**Cleaning Instructions** (as needed)

1. Wipe monitor plug and cable with water or alcohol.
2. Clean outside of catheter connector with soap and water.
3. Apply water or alcohol to a swab (included).
4. Insert swab inside cable connector cone as shown below.
5. Rotate swab and move it around inside cone.



**DO NOT SUBMERGE OR SOAK CABLE CONNECTOR!**  
If swabs other than those included are used, take care to avoid disturbing the transducer protecting gel inside the connector.

**REF** Catalog No.: **IPC-SWAB**

P/N 050-0096 Rev. D



CLINICAL  
INNOVATIONS

747 West 4170 South  
Murray, Utah 84123  
Phone 888-268-6222  
Fax 801-266-7373  
[www.clinicalinnovations.com](http://www.clinicalinnovations.com)

Figure G